

Urine Sample Collection Instructions

For Patients

Patient Collection Instructions: Routine Urinalysis



NOTE: Caution do not to remove the safety label on the lid to protect against needle sticks from the "sharp" contained in the integrated transfer device.

1. Complete the patient information on the labels that is included on the ToxKit requisition form and attach it to the urine collection cup.
2. Wash hands thoroughly with soap and water.



3. Remove collection cup lid and place lid on counter with "threaded" portion of lid facing upwards. **Do not touch inside of lid.**
4. Cleanse the area around the urethral opening with the castile soap towelette.
5. Urinate into the urine collection Cup without touching the inside or lip of the cup. Fill collection cup to first line or above.



6. Replace the lid onto the collection cup.
7. Return the sample to the health care worker for processing.

PRESCIENT MEDICINE

1214 Research Blvd, Suite 1000, Hummelstown, PA 17036 USA
+1 844 718 1600 toll-free +1 717 220 7099 fax
prescientmedicine.com



Urine Sample Collection Instructions

For Healthcare Professionals

Healthcare Worker Collection Instructions: Routine Urinalysis



1. Remove safety label from integrated transfer device on lid of urine collection cup.



2. Insert the VACUETTE® Tube into the integrated transfer device with the safety cap down. Ensure that the needle penetrates the stopper of the urine tube. Urine will flow automatically in accordance to the predefined vacuum within the tube.



3. Once tube is done filling, remove tube and label specimen immediately with the label included on the ToxKit requisition form. (Make sure the sample ID matches the ID on the urine cup and that there are 2 identifiers on the label, such as Patient Name and DOB).



4. Dispose of urine collection cup.
Remember lid contains a needle in the integrated transfer device.

Shipping Instructions

1. Complete the ToxKit requisition form.
2. Confirm that the patient information on the Urine Collection Cup or VACUETTE Tube matches the information on the ToxKit Requisition Form. Place VACUETTE Tube into the absorbant sleeve.
3. Place the Urine Collection Cup or VACUETTE Tube and ToxKit requisition form into the biohazard bag, seal the bag and either place in area for courier pick-up or ship to Prescient Medicine with FedEx Clinical Pak.

PRESCIENT MEDICINE

1214 Research Blvd, Suite 1000, Hummelstown, PA 17036 USA
+1 844 718 1600 toll-free +1 717 220 7099 fax
prescientmedicine.com

